Beipackzettel / Notice pour le public / User information  
Stampato informativo / Bjsluitter / Informacja dla użytkownika  
Informácia pre používateľov

# Helipur®

**Desinfektionsmittel** selbsttätig reinigend  
**Désinfectant** à haut pouvoir nettoyant

**Disinfectant** with powerful cleaning action

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Active against bacteria, TbB, fungi,  
enveloped viruses (incl. HBV, HCV, HIV)\*

**Zusammensetzung / Composition / Ingrédients / Composizione / Samenstelling**  
**Składniki / Zloženie:** 100 g solution contains Chlorocresol 8,5 g,  
Clorofen 4,8 g, Biphenyl-2-ol 4,0 g. Excipients: >30 % anionic  
surfactants, 15-30 % solvents, corrosion inhibitors, <5 % phosphonates,  
sodium hydroxide, corrosion inhibitors, parfum, colourants.

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Ⓜ **Use:** Disinfection and cleaning of instruments, laboratory equipment,  
surfaces and excrements. Suitable for ultrasound cleaning.

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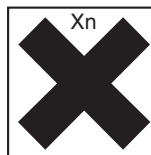
Ⓜ **After disinfection, rinse instruments thoroughly with water and proceed as  
required. Not compatible with materials sensitive to alkaline, phenol and  
thermolabile instruments (e.g. endoscopes).**

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Ⓜ **Flammable.** Harmful in contact with skin and if swallowed. Irritating to skin. Risk of  
serious damage to eyes. May cause sensitization by skin contact. Toxic to aquatic orga-  
nisms, may cause long-term adverse effects in the aquatic environment. In case of contact  
with eyes rinse immediately with plenty of water and seek medical advice. This material  
and its container must be disposed of in a safe way. Wear suitable protective clothing, glo-  
ves and eye/face protection. Avoid release to the environment. Refer to special instructions/  
safety data sheets. Keep out of reach of children.

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**B. Braun Melsungen AG**
**B. Braun Medical AG**